

Photo mirror

Instructions for use

With the purchase of this photo mirrors you receive a high-quality product that was developed in close cooperation with professional institutes and has been established on the world market for over 35 years. The very hard and scratch-resistant mirror coating to the rounded mirror edge guarantees

- high reflection
- good colour neutrality
- no double images

The angled handle allows the mirror to be held without affecting the mirror surface.

Please read these instructions for use carefully and keep them in a suitable place. Strict compliance with our instructions guarantees hazard-free, safe handling of our product as well as the longest possible service life.

Materials:

The mirror is made of glass coated with rhodium, which is glued manually to the handle. The handle is cast aluminium coated with silicone polyester. The adhesive used is an anaerobic and UV light-curing one-component resin without solvent.

Scope of validity:

This document is valid for the following photo mirrors:

- Types A, B1, B2 & C, for occlusal photographs
- Type D, for lateral photographs
- Type S, mirror set
- Type O, for occlusal photographs without handle
- Type L, for lingual photographs without handle



Mirrors small, medium, large and lateral

Intended use:

The mirrors are intended for photographic documentation during dental examinations. The photographs are used for treatment planning, reassessment during treatment and final and late findings.

Safe handling and provision:

- Use and preparation by trained personnel only
- The mirrors must not be used contrary to their intended purpose, as this may result in damage or breakage
- Clean, disinfect and/or sterilize the product before first use
- Store products in such a way that they are not damaged
- Do not use battered products and discard defective products

Reprocessing instructions:

- As an acrylate, the anaerobic adhesive used is not permanently solvent-resistant; through the effect of cleaning or disinfection solutions, the adhesive bond is damaged, and liquid penetrates the glue line. The mirrors may not therefore be immersed for a long period. The manufacturer does not assume any warranty for this
- Comply with international and national guidelines, standards and legal regulations as well as your own hygiene regulations for reprocessing
- For persons with infectious diseases, corresponding guidelines must be followed

Restriction of reprocessing:

Extreme care must be taken when reprocessing due to the thin mirror coating and the lacquered handles. Frequent, but proper reprocessing has only little effect on the quality and service life of the mirrors.

The end of the product service life is normally determined by wear and tear through use. The mirrors must then be disposed of in the usual manner in dental practices.

FILTROP AG

Landstrasse 16
9496 Balzers | Liechtenstein
Phone +423 388 11 50
info@filtrop.com
www.filtrop.com

Disinfection:

After use, the mirrors can be

- cleaned manually by means of wipe disinfection
- immersed in a mild cleaning or disinfection solution suitable for instruments

The cleaning or disinfection solution should be suitable for manual cleaning (e.g. non-foaming).

Change the cleaning or disinfection solution according to the manufacturer's instructions or in the case of visible contamination.

Ultrasonic cleaning is **not** recommended.

In general, clean and disinfect the mirrors according to the recommendations of the Robert Koch Institute.

Neutralisation / clear rinsing:

Rinse cleaned or disinfected mirrors according to standard practice with clean or sterile water to remove all cleaning and disinfection residues.

Sterilisation:

The mirrors can be sterilised, for example in an autoclave.

Avoid sterilisation liquids containing aggressive chemicals such as hypochlorite, however.

After frequent sterilisation, wear and tear must be anticipated (e.g. partial detachment of the lacquer coating on the handle).

Recommended sterilisation method:

In an autoclave (DIN EN ISO 17665-1)

Recommended temperature: 121 °C (134 °C)

Recommended pressure: 2 bar (3 bar)

Holding time: ≥ 20 min (≥ 5 min)

Drying and storage:

After the neutralisation, allow the mirrors to air dry on a clean cloth at room temperature or dry with compressed air. Keep the dried mirror clean and closed in its original packaging to prevent re-contamination.

Functional testing:

Check the functionality of the mirrors before each use. Products with damage to the glass surfaces and the coatings (scratches, chipping, cracks, etc.) may no longer be used.

The mirrors must then be sent for disposal in accordance with the standard practice.

Risks:

The following risks may occur:

- Cutting injuries if the mirror is damaged or in people who have seizures, due to splinters and sharp edges
- Allergic reactions:
 - to aluminium, if handles are damaged
 - to the adhesives, if not properly prepared

Restriction of use:

For persons with seizure susceptibilities, the mirrors should only be used together with jaw locks.

For persons with infectious diseases, the mirrors with handles should not be used. We advise the use of mirrors without handles instead:



Mirrors without handle for occlusal and lingual photographs

Warranty:

The mirrors are made of high-quality materials and undergo quality control before delivery. We do not guarantee the suitability for the respective operation. We also accept no liability for accidental or resulting damage. We accept no liability if it can be proven that these instructions for use have been violated in particular in the case of improper handling of the cleaning / disinfection.

FILTROP AG

Landstrasse 16
9496 Balzers | Liechtenstein
Phone +423 388 11 50
info@filtrop.com
www.filtrop.com